×

**MEDIAID**

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# Soft-Power

## INSTRUCTIONS FOR USE

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use.  
Also store it in a location where you can refer to it at any time.

### **[Read this first]**

- Support may not be provided by this product under certain usage conditions.
- This product is not intended for therapeutic use.
- This product will not enable you to exert more than your normal strength. Avoid lifting excessively heavy objects or postures that impose more burden than necessary.

### **[When using the product]**

- Because of the characteristics of the material, the color may change under certain storage conditions.
- Do not compress the product strongly during storage. Strong compression may cause the hook-and-loop fasteners to adhere less firmly to the belt.

## [Precautions for use – Be sure to read this]

Safety precautions are divided into the following sections, according to the level of danger involved.

**⚠ WARNING ...** Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

**⚠ CAUTION ...** Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

### ⚠ WARNING


- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product.
  - Those predisposed to allergies or with sensitive skin.
  - Those who have a fracture, cut, numbness, swelling, eczema or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
  - Do not use this product for purposes other than work assistance.
  - Be sure to use the correct size product.
  - If you are pregnant or you think you are pregnant, consult a physician.
  - Use properly according to the Wearing Procedure.
  - Do not overtighten the belts.
  - If the product becomes loose or misaligned during use, repeat the wearing procedure from the beginning to ensure that you are wearing it properly. When wearing the product for a long time, be alert to excessive compression, etc. If necessary, release fasteners and wear it again.
  - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
  - Do not attempt to modify or repair the product.
  - Do not use if elastic parts or fabric if they no longer stretch.
- If any abnormality such as injury, fracture, pain, numbness, swelling, or eczema occurs while using this product, stop using it immediately and consult a physician. Your symptoms may worsen if you continue to use the product.

### ⚠ CAUTION


- When buckling and unbuckling, be careful not to pinch your fingers.
- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Be sure to wear the product over an undershirt or similar garment instead of directly on your skin.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners do not attach to clothing. If they do, clothing may become frayed or be damaged.

## [Names of the parts]

### < Front >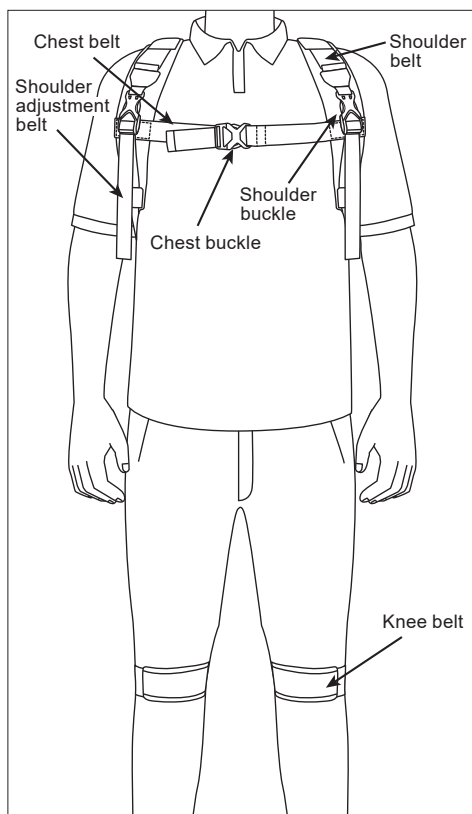



### < Back >

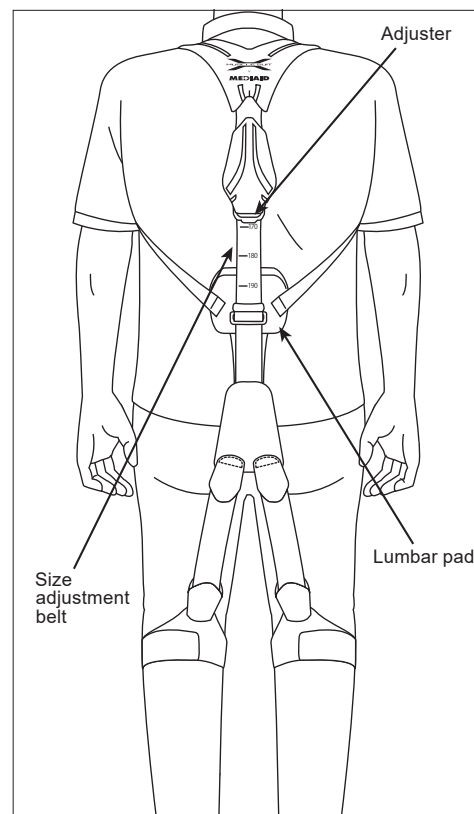
## [Wearing Procedure]

- Before wearing this product, carefully read the "Precautions for use - Be sure to read this" section of this Instructions for Use. Then follow the steps provided to ensure the product is correctly worn.
- Confirm the top/bottom, inside/outside, and front/back of the product. Release the hook-and-loop fasteners of the knee belts, and loosen the shoulder belts.


- 1** Use the size adjustment belt to adjust to your body height. The memory shows body height in centimeters. Slide the adjuster to your body height.
- 2** Pass your arms through the shoulder belts.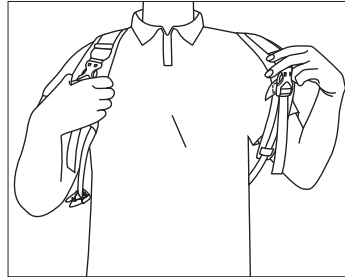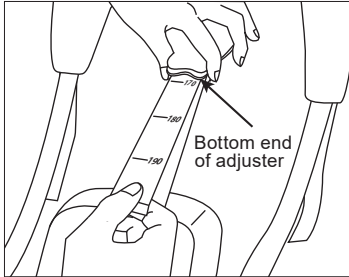


- 3** Close the chest buckle.

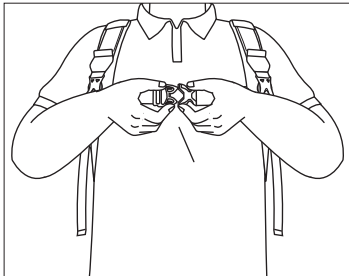
- 4** Release the shoulder buckles.

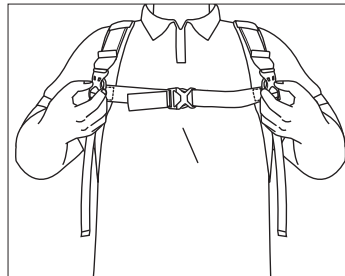
- 5** Close the hook-and-loop fasteners to secure the knee belts. Each belt should be positioned under the kneecap.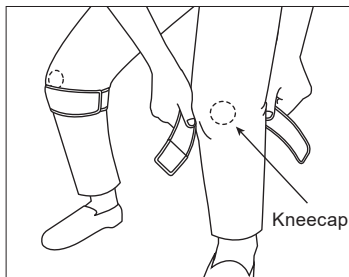



- 6** Close the shoulder buckles.

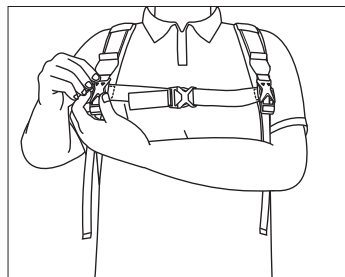
- 7** Pull and adjust the chest belt and shoulder adjustment belts to fit the body. Pull the shoulder adjustment belt downward. The position of the chest belt can be adjusted.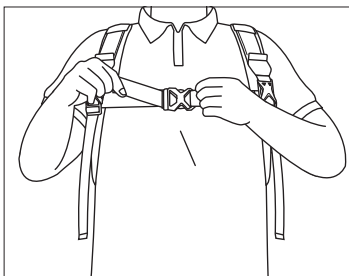

- 8** Make sure that the product is not loose or out of place. The lumbar pad should be positioned at about the level of the navel. If you feel any discomfort, repeat the steps from the beginning to reattach the product.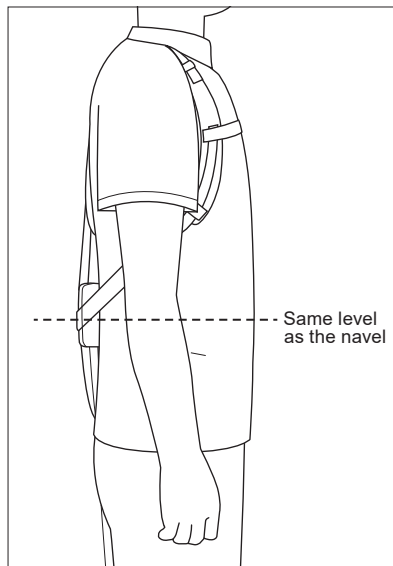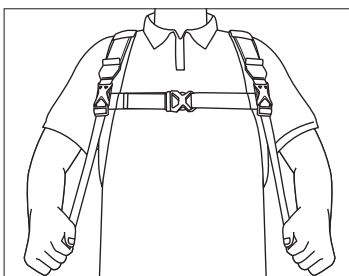

## [Care and clean]

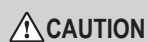- Do not wash the product with light colored items. By doing so, color may fade or be transferred to other materials.

- This product can be washed in the washing machine. Wash in a laundry bag. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use iron or use chlorine bleach. However, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

## [Size table]

Size	Range (Body height)
One size fits all	150 cm - 190 cm

## [Reporting Obligation]

If you have any complaints about the product, please contact your distributor directly.

It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

- \* When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- \* The product specifications and appearance are subject to change without notice for the purpose of improvement.
- \* We take all possible measures to ensure product quality, however, if you find any defects, etc., in our products, please contact sales agent.

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Innophys Co., Ltd.
NIPPON SIGMAX CO., LTD